Augentropfen

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Similasan

Similasan

Augen Gel Tropfen

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Ingredients

Sodium hyaluronate 3 mg/ml, sodium chloride, sodium hydrogen phosphate, chlorides (potassium, calcium, magnesium), sodium hydrogen carbonate, purified water.

Areas of application

For intense and sustained lubrication of the eye surface in case of dry, irritated or burning eyes and for the relief and treatment of irritations due external factors (such as a dry or air-conditioned environment, smoke, dust, pollen) as well as after surgery on the eye. Similasan Gel eye drops can also be used with hard or soft contact lenses.

Contraindications

Similasan Gel eye drops should not be used in the presence of a hypersensitivity to any of the ingredients.

Notes

- Please note that your vision may be temporarily affected immediately following the use of the product.
- Do not use Similasan Gel eye drops if the bottle has been damaged.
- Do not use Similasan Gel eye drops simultaneously with other eye medicines because the effect of the preparations may be altered. In principle, if other eye drops / ointments are used, an interval of approx. 30 minutes between using the different products should be adhered to.
- In order to avoid contamination of the external surface, please ensure that you avoid touching the surface of the eye or other items with the tip of the dropper.
- In very rare cases, calcification of the cornea in association with the use of eye drops with a high phosphate concentration has been reported in patients with significant damage to the cornea. The eye drops contain only a physiological concentration of phosphate and so their application, in principle, is not restricted to existing severe corneal disorders / damage only. If other eye drop preparations are also used, however, their phosphate concentration should be borne in mind in this regard since, particularly in the case of anti-glaucomatous agents or anti-allergy agents, they may contain high phosphate concentrations.

Tips for use

- Drop Similasan Gel eye drops onto the surface of the eye or into the conjunctival sac.
- Washing hands thoroughly before use is recommended.
 - Before using the eye drops, remove the protective cap from the tip of the bottle whenever you use it. Make sure that you do not touch the tip of the dropper with your fingers (See Fig. 1).
 - Place your thumb on the ergonomically formed thumb rest and wrap your other fingers around the lower part of the bottle (See Fig. 2).
 - Turn the bottle upside down and hold it as shown in the illustration. If you are using the pump for the first time, you will first need to perform a few pressing movements before the first drop appears (See Fig. 3).
 - Tilt your head back slightly and hold the bottle vertically above your eye. Pull your lower eyelid down with one of your fingers and hold the bottle vertical while carefully pressing against the thumb rest to apply a drop onto your eye (See Fig. 4).
 - To prevent the risk of contaminating the solution, make sure that the dropper tip does not come into contact with your eyes or fingers (See Fig. 4).
 - Once you have finished, always place the protective cap straight back on the dropper tip (See Fig. 5).

Directions for use

Similasan Gel eye drops can be used as required. In general, application up to 4 times per day is recommended.

Side effects

Hypersensitivity reactions may occur in very rare cases. In this case, the product should not be used again.

Shelf-life and storage

Store in a cool and dry place. Keep out of reach of children. Can be used for up to 6 months after first use. Do not use after the expiry date.

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